
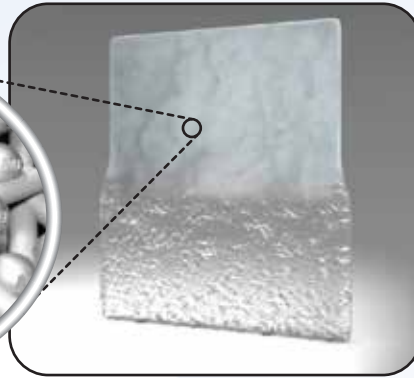


Fibrosol® Ag Family Dressings

Fibrosol with Silver Technology

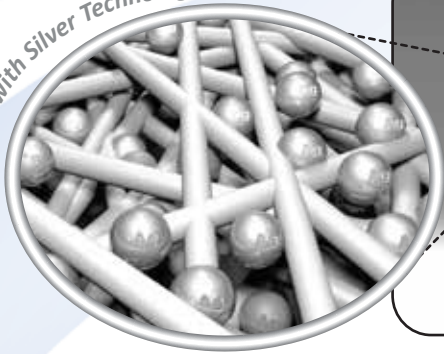

Description:

Fibrosol® Ag: are primary wound dressing made of carboxy methyl cellulose nonwoven containing silver.

- The characteristics of the fiber are such that it absorbs when it comes into intimate contact with liquid not via the usual mechanism of capillary action. Absorbed fluid is retained within the structure of the fiber. As **Fibrosol® Ag** absorbs exudate it is rapidly converted from a dry dressing to a soft coherent gel that maintains a moist wound environment for optimal wound healing, aids autolytic debridement, and it is easily removed with little or no damage to newly formed tissue.
- The presence of silver in the dressing provides a broad spectrum antimicrobial activity against many gram positive, gram negative bacteria and fungi.
- **Fibrosol® Ag Dressings** have five product lines **Fibrosol® Ag pad**, **Fibrosol® Ag Island dressings**, **Fibrosol® Ag OTC bandages**, **Fibrosol® Ag Foam Gentle** and **Fibrosol® Ag Foam Basic**.

1. Non-Adherent Fibrosol® Ag Wound Pad variants:

A. Fibrosol® Ag Non-Adhesive: Antimicrobial primary absorbent wound pad that is cut into different size, it needs a secondary dressing for fixation. Available as flat sheets loose or stitch bonded for support upon gelling or in the form of ribbon for tunnelling and cavity wounds.

B. Fibrosol® Ag Plus Non-Adhesive: Antimicrobial primary absorbent wound pad with the addition of Benzethonium chloride, which lowers biofilms' surface tension to improve EDTA's ability to remove metal ions from them and exposes microorganisms to the antimicrobial effect of silver.

C. Fibrosol® Ag Extra Non-Adhesive: is made up of 2 layers of **Fibrosol® Ag pad** stitch bonded together providing extra absorption and strength properties.

D. Fibrosol® Ag Plus Extra Non-Adhesive: is made up of 2 layers of **Fibrosol® Ag Plus I** stitch bonded together providing extra absorption and strength properties

2. Fibrosol® Ag island dressings and Fibrosol® Ag OTC bandage:

The pad is placed centrally on an adhesive substrate that extends beyond the border of the pad to allow fixation without secondary dressing.

A. Fibrosol® Island dressings: Island dressing could be any of the following sub-variants:

I. Fibrosol® Ag adhesive or Fibrosol® Ag Extra adhesive: Soft conformable post-operative non-woven dressing with highly absorbent antimicrobial pad **Fibrosol® Ag Non Adhesive or Fibrosol® Ag Extra Non Adhesive**.


II. Fibrosol® Ag adhesive Film and Fibrosol® Ag Extra adhesive Film: Transparent, breathable, virus proof, bacteria proof and waterproof polyurethane film with highly absorbent antimicrobial **Fibrosol® Ag Non Adhesive or Fibrosol® Ag Extra Non Adhesive**.

Fibrosol® Ag adhesive Film Frame style and Fibrosol® Ag adhesive Film Extra Frame Style is equipped with frame delivery system for easy handling of the dressing. Indicated when waterproofness and bacterial and viral barrier properties are essential.

III. Fibrosol® Ag adhesive Ultra or Fibrosol® Ag Extra adhesive Ultra: Nonwoven dressing coated with breathable waterproof, virus proof and bacteria proof polyurethane film outermost layer and central highly absorbent antimicrobial pad **Fibrosol® Ag Non Adhesive or Fibrosol® Ag Extra Non Adhesive**.

IV. Fibrosol® Ag IV or Fibrosol® Ag Extra IV: Non-woven I.V. catheter fixation dressing for people with sensitive skin, having incision along the middle and highly absorbent antimicrobial pad at the end of Incision, located in central position. Indicated for cannula and catheter fixation on sensitive skin.

V. Fibrosol® Ag IV film or Fibrosol® Ag Extra IV film: Transparent, breathable, virus proof, waterproof and bacteria proof polyurethane I.V. dressing, having incision along the middle and highly absorbent antimicrobial pad at the end of incision, located in central position. Indicated to cannula and catheter fixation.

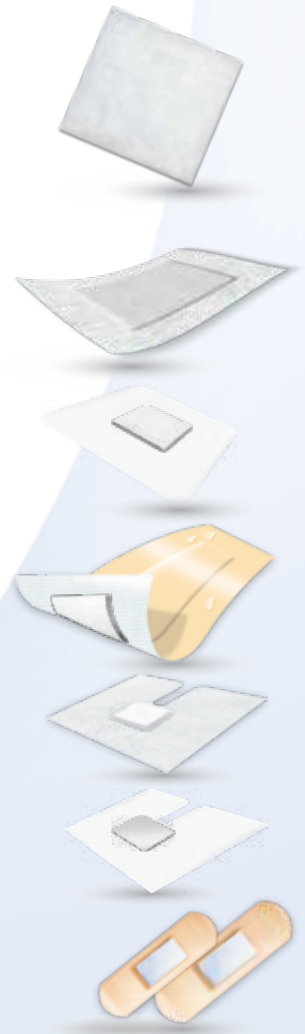

Fibrosol® Ag Family Dressings

B. Fibrosol® Ag OTC Bandage:

I. **Fibrosol® Ag Adhesive bandages** for bleeding minor cuts, scratches and abrasions.

II. **Fibrosol® Ag Dressing strip** Adhesive bandages for bleeding minor cuts, scratches and abrasions. It could be cut to size.

3. Fibrosol® Ag Foam Dressings:

I. **Fibrosol® Ag Foam Gentle or Fibrosol® Ag Extra Foam Gentle:** is composed of soft absorbent polyurethane foam laminated to a **Fibrosol® Ag Non adhesive or Fibrosol® Ag Extra Non adhesive** layer placed as an island pad under a waterproof, virus proof and bacteria proof polyurethane film coated with silicone adhesive that allow gentle dressing removal without pain.

The silicone adhesive layer is present either as a border surrounding the dressing or as a perforated layer covering the whole dressing.

II. **Fibrosol® Ag Foam Basic or Fibrosol® Ag Extra Foam Basic:** is composed of soft absorbent polyurethane foam laminated to a **Fibrosol® Ag Non adhesive or Fibrosol® Ag Extra Non adhesive** layer placed as an island pad under a waterproof, virus proof and bacteria proof polyurethane film coated with hydrocolloid adhesive which turns into gel upon absorption so maintains a moist wound environment.

The hydrocolloid layer is present either as a border surrounding the dressing or as a perforated layer covering the whole dressing.

Features:

- Provides excellent absorption and retention capabilities for moderate to highly exuding wounds.
- Conforms to the wound surface to form an intimate contact.
- Creates a moist environment that aids wound healing.
- Acts as a bacterial barrier that prevents outer wound infection.
- Helps reduce wound pain while the dressing is in situ and upon removal.
- Supports wound healing by providing a moist wound healing environment.
- Hemostatic effect.
- Broad antimicrobial activity.

Indications:

» **Fibrosol® Ag dressings can be used in the management of:**

- Infected wounds and those of risk of infection.
- Second degree burns.
- Partial thickness burns (Fibrosol® Ag Plus Extra Non-Adhesive)
- Malignant wounds
- Diabetic foot ulcers, pressure ulcers and leg ulcers.
- Surgical wounds.
- Traumatic wounds.
- Wounds those are prone to bleeding.
- Oncology wounds.
- Donor and recipient graft sites.
- Treatment of primary infected wounds
- Effective bacterial barrier that reduces wound infection
- Pressure sores
- Leg ulcers
- Ischaemic and diabetic ulcers
- Post-operative wounds
- skin graft donor sites
- Fungating carcinomas

» **Fibrosol® Ag OTC bandages:** are used for cuts, grazes, minor abrasions and to control bleeding minor wounds. Also **Fibrosol® Ag adhesive bandages** are used to achieve haemostasis at the bleeding site after needle removal in the renal dialysis and for protection of orificium fistulae.

» **Fibrosol® Ag IV dressings** are used for catheter and cannula fixation, Fibrosol pad provides an extra stop bleeding action.


» **Fibrosol® Ag Ribbon** can be used for tunneling and cavity wounds.

Contraindications:


- **Fibrosol® Ag dressings** should not be used on people who are allergic to any of the dressing's components.
- Experience is limited with newly born, infants and during pregnancy.

Precautions:

- **Fibrosol® Ag** is not intended for use within internal body cavities.
- If infection develops during the use of the dressing, clinically indicated antibiotic therapy should be initiated.
- Control of blood glucose should be monitored for diabetic foot ulcers.
- Delicate newly formed blood vessels could produce blood stained wound fluid.
- Sterility is guaranteed unless pouch is damaged.

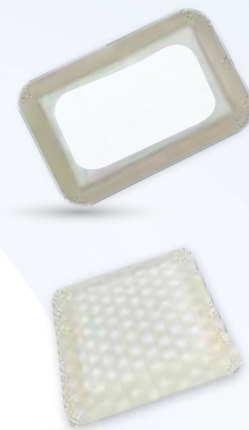

Fibrosol® Ag Family Dressings

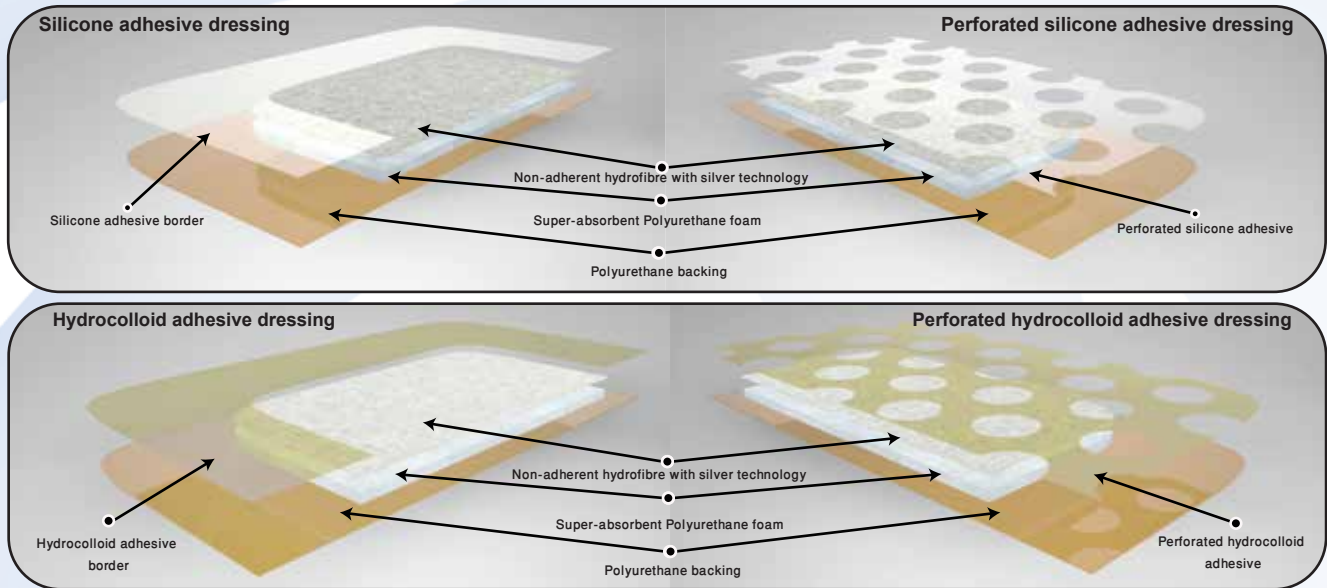

Sizes

Fibrosol® Ag Non Adhesive	Fibrosol® Ag Extra Non Adhesive	cm	inch	Pieces/Box	Boxes/Case
FIBAG5050	FIBEXAG5050	5x5cm			
FIBAG100100	FIBEXAG100100	10x10cm			
FIBAG150150	FIBEXAG150150	15x15cm			
FIBAG100150	FIBEXAG100150	10x15cm			
FIBAG100200	FIBEXAG100200	10x20cm			
FIBAG200450	FIBEXAG200450	20x45cm			
FIBAG5070	FIBEXAG5070	5x7cm			
FIBAG100250	FIBEXAG100250	10x25cm			
FIBAG100300	FIBEXAG100300	10x30cm			
FIBAG6070	FIBEXAG6070	6x7cm			
FIBAG100120	FIBEXAG100120	10x12cm			
FIBAG6080	FIBEXAG6080	6x8cm			
FIBAG165180	FIBEXAG165180	16.5x18cm			
FIBAG220220	FIBEXAG220220	22x22cm			
FIBAG25450	FIBEXAG25450	2.5x45cm			
FIBAG150200	FIBEXAG150200	15x20cm			
FIBAG150250	FIBEXAG150250	15x25cm			
FIBAG20300	FIBEXAG20300	2x30cm			
FIBAG2040	FIBEXAG2040	2x4cm			
FIBAG200200	FIBEXAG200200	20x20cm			
FIBAG200300	FIBEXAG200300	20x30cm			
FIBAG50100	FIBEXAG50100	5x10cm			
FIBAG50300	FIBEXAG50300	5x30cm			
FIBAG5075	FIBEXAG5075	5x7.5cm			
FIBAG70120	FIBEXAG70120	7x12cm			
FIBAG75100	FIBEXAG75100	7.5x10cm			
FIBAG75120	FIBEXAG75120	7.5x12cm			
FIBAG75150	FIBEXAG75150	7.5x15cm			
FIBAG75200	FIBEXAG75200	7.5x20cm			
FIBAG7575	FIBEXAG7575	7.5x7.5cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag adhesive	Fibrosol® Ag Extra adhesive	cm	inch	Pieces/Box	Boxes/Case
FIBADAG5070	FIBADEXAG5070	5x7cm			
FIBADAG100100	FIBADEXAG100100	10x10cm			
FIBADAG150150	FIBADEXAG150150	15x15cm			
FIBADAG100150	FIBADEXAG100150	10x15cm			
FIBADAG100200	FIBADEXAG100200	10x20cm			
FIBADAG200450	FIBADEXAG200450	20x45cm			
FIBADAG100250	FIBADEXAG100250	10x25cm			
FIBADAG100300	FIBADEXAG100300	10x30cm			
FIBADAG6070	FIBADEXAG6070	6x7cm			
FIBADAG100120	FIBADEXAG100120	10x12cm			
FIBADAG6080	FIBADEXAG6080	6x8cm			
FIBADAG165180	FIBADEXAG165180	16.5x18cm			
FIBADAG220220	FIBADEXAG220220	22x22cm			
FIBADAG5090	FIBADEXAG5090	5x9cm			
FIBADAG6083	FIBADEXAG6083	6x8.3cm			
FIBADAG50100	FIBADEXAG50100	5x10cm			
FIBADAG6082	FIBADEXAG6082	6x8.25cm			
FIBADAG6085	FIBADEXAG6085	6x8.5cm			
FIBADAG7080	FIBADEXAG7080	7x8cm			
FIBADAG6580	FIBADEXAG6580	6.5x8cm			
FIBADAG7575	FIBADEXAG7575	7.5x7.5cm			
FIBADAG6090	FIBADEXAG6090	6x9cm			
FIBADAG60100	FIBADEXAG60100	6x10cm			
FIBADAG5082	FIBADEXAG5082	5x8.25cm			
FIBADAG75100	FIBADEXAG75100	7.5x10cm			
FIBADAG80100	FIBADEXAG80100	8x10cm			
FIBADAG82100	FIBADEXAG82100	8.25x10cm			
FIBADAG70120	FIBADEXAG70120	7x12cm			
FIBADAG80120	FIBADEXAG80120	8x12cm			
FIBADAG83120	FIBADEXAG83120	8.3x12cm			
FIBADAG90150	FIBADEXAG90150	9x15cm			
FIBADAG80150	FIBADEXAG80150	8x15cm			
FIBADAG82150	FIBADEXAG82150	8.25x15cm			
FIBADAG100140	FIBADEXAG100140	10x14cm			
FIBADAG83180	FIBADEXAG83180	8.3x18cm			
FIBADAG82200	FIBADEXAG82200	8.25x20cm			
FIBADAG90200	FIBADEXAG90200	9x20cm			
FIBADAG200200	FIBADEXAG200200	20x20cm			
FIBADAG83240	FIBADEXAG83240	8.3x24cm			
FIBADAG82250	FIBADEXAG82250	8.25x25cm			
FIBADAG90250	FIBADEXAG90250	9x25cm			
FIBADAG82300	FIBADEXAG82300	8.25x30cm			
FIBADAG90300	FIBADEXAG90300	9x30cm			
FIBADAG90350	FIBADEXAG90350	9x35cm			
FIBADAG82350	FIBADEXAG82350	8.25x35cm			
FIBADAG100350	FIBADEXAG100350	10x35cm			
FIBADAG83415	FIBADEXAG83415	8.3x41.5cm			
FIBADAG100160	FIBADEXAG100160	10x16cm			
FIBADAG100210	FIBADEXAG100210	10x21cm			
FIBADAG100260	FIBADEXAG100260	10x26cm			
FIBADAG100400	FIBADEXAG100400	10x40cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag adhesive	Fibrosol® Ag Extra adhesive	cm	inch	Pieces/Box	Boxes/Case
FIBADAG120160	FIBADEXAG120160	12x16cm			
FIBADAG72250	FIBADEXAG72250	7.2x25cm			
FIBADAG5072	FIBADEXAG5072	5x7.2cm			
FIBADAG5075	FIBADEXAG5075	5x7.5cm			
FIBADAG6075	FIBADEXAG6075	6x7.5cm			
FIBADAG70100	FIBADEXAG70100	7x10cm			
FIBADAG85155	FIBADEXAG85155	8.5x15.5cm			
FIBADAG85150	FIBADEXAG85150	8.5x15cm			
FIBADAG8595	FIBADEXAG8595	8.5x9.5cm			
FIBADAG90100	FIBADEXAG90100	9x10cm			
Fibrosol® Ag adhesive Film	Fibrosol® Ag Extra adhesive Film	cm	inch	Pieces/Box	Boxes/Case
FIBPUADAG5070	FIBPUADEXAG5070	5x7cm			
FIBPUADAG100100	FIBPUADEXAG100100	10x10cm			
FIBPUADAG150150	FIBPUADEXAG150150	15x15cm			
FIBPUADAG100150	FIBPUADEXAG100150	10x15cm			
FIBPUADAG100200	FIBPUADEXAG100200	10x20cm			
FIBPUADAG200450	FIBPUADEXAG200450	20x45cm			
FIBPUADAG100250	FIBPUADEXAG100250	10x25cm			
FIBPUADAG100300	FIBPUADEXAG100300	10x30cm			
FIBPUADAG6070	FIBPUADEXAG6070	6x7cm			
FIBPUADAG100120	FIBPUADEXAG100120	10x12cm			
FIBPUADAG6080	FIBPUADEXAG6080	6x8cm			
FIBPUADAG165180	FIBPUADEXAG165180	16.5x18cm			
FIBPUADAG220220	FIBPUADEXAG220220	22x22cm			
FIBPUADAG5090	FIBPUADEXAG5090	5x9cm			
FIBPUADAG6083	FIBPUADEXAG6083	6x8.3cm			
FIBPUADAG50100	FIBPUADEXAG50100	5x10cm			
FIBPUADAG6082	FIBPUADEXAG6082	6x8.25cm			
FIBPUADAG6085	FIBPUADEXAG6085	6x8.5cm			
FIBPUADAG7080	FIBPUADEXAG7080	7x8cm			
FIBPUADAG6580	FIBPUADEXAG6580	6.5x8cm			
FIBPUADAG7575	FIBPUADEXAG7575	7.5x7.5cm			
FIBPUADAG6090	FIBPUADEXAG6090	6x9cm			
FIBPUADAG60100	FIBPUADEXAG60100	6x10cm			
FIBPUADAG5082	FIBPUADEXAG5082	5x8.25cm			
FIBPUADAG75100	FIBPUADEXAG75100	7.5x10cm			
FIBPUADAG80100	FIBPUADEXAG80100	8x10cm			
FIBPUADAG82100	FIBPUADEXAG82100	8.25x10cm			
FIBPUADAG70120	FIBPUADEXAG70120	7x12cm			
FIBPUADAG80120	FIBPUADEXAG80120	8x12cm			
FIBPUADAG83120	FIBPUADEXAG83120	8.3x12cm			
FIBPUADAG90150	FIBPUADEXAG90150	9x15cm			
FIBPUADAG80150	FIBPUADEXAG80150	8x15cm			
FIBPUADAG82150	FIBPUADEXAG82150	8.25x15cm			
FIBPUADAG100140	FIBPUADEXAG100140	10x14cm			
FIBPUADAG83180	FIBPUADEXAG83180	8.3x18cm			
FIBPUADAG82200	FIBPUADEXAG82200	8.25x20cm			
FIBPUADAG90200	FIBPUADEXAG90200	9x20cm			
FIBPUADAG200200	FIBPUADEXAG200200	20x20cm			
FIBPUADAG83240	FIBPUADEXAG83240	8.3x24cm			
FIBPUADAG82250	FIBPUADEXAG82250	8.25x25cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag adhesive Film	Fibrosol® Ag Extra adhesive Film	cm	inch	Pieces/Box	Boxes/Case
FIBPUADAG90250	FIBPUADEXAG90250	9x25cm			
FIBPUADAG82300	FIBPUADEXAG82300	8.25x30cm			
FIBPUADAG90300	FIBPUADEXAG90300	9x30cm			
FIBPUADAG90350	FIBPUADEXAG90350	9x35cm			
FIBPUADAG82350	FIBPUADEXAG82350	8.25x35cm			
FIBPUADAG100350	FIBPUADEXAG100350	10x35cm			
FIBPUADAG83415	FIBPUADEXAG83415	8.3x41.5cm			
FIBPUADAG100160	FIBPUADEXAG100160	10x16cm			
FIBPUADAG100210	FIBPUADEXAG100210	10x21cm			
FIBPUADAG100260	FIBPUADEXAG100260	10x26cm			
FIBPUADAG100400	FIBPUADEXAG100400	10x40cm			
FIBPUADAG120160	FIBPUADEXAG120160	12x16cm			
FIBPUADAG72250	FIBPUADEXAG72250	7.2x25cm			
FIBPUADAG5072	FIBPUADEXAG5072	5x7.2cm			
FIBPUADAG5075	FIBPUADEXAG5075	5x7.5cm			
FIBPUADAG6075	FIBPUADEXAG6075	6x7.5cm			
FIBPUADAG70100	FIBPUADEXAG70100	7x10cm			
FIBPUADAG85155	FIBPUADEXAG85155	8.5x15.5cm			
FIBPUADAG85150	FIBPUADEXAG85150	8.5x15cm			
FIBPUADAG8595	FIBPUADEXAG8595	8.5x9.5cm			
FIBPUADAG90100	FIBPUADEXAG90100	9x10cm			
Fibrosol® Ag adhesive Film Frame Style	Fibrosol® Ag Extra adhesive Film Frame style	cm	inch	Pieces/Box	Boxes/Case
FIBPUFSADAG5070	FIBPUFSADEXAG5070	5x7cm			
FIBPUFSADAG100100	FIBPUFSADEXAG100100	10x10cm			
FIBPUFSADAG150150	FIBPUFSADEXAG150150	15x15cm			
FIBPUFSADAG100150	FIBPUFSADEXAG100150	10x15cm			
FIBPUFSADAG100200	FIBPUFSADEXAG100200	10x20cm			
FIBPUFSADAG200450	FIBPUFSADEXAG200450	20x45cm			
FIBPUFSADAG100250	FIBPUFSADEXAG100250	10x25cm			
FIBPUFSADAG100300	FIBPUFSADEXAG100300	10x30cm			
FIBPUFSADAG6070	FIBPUFSADEXAG6070	6x7cm			
FIBPUFSADAG100120	FIBPUFSADEXAG100120	10x12cm			
FIBPUFSADAG6080	FIBPUFSADEXAG6080	6x8cm			
FIBPUFSADAG165180	FIBPUFSADEXAG165180	16.5x18cm			
FIBPUFSADAG220220	FIBPUFSADEXAG220220	22x22cm			
FIBPUFSADAG5090	FIBPUFSADEXAG5090	5x9cm			
FIBPUFSADAG6083	FIBPUFSADEXAG6083	6x8.3cm			
FIBPUFSADAG50100	FIBPUFSADEXAG50100	5x10cm			
FIBPUFSADAG6082	FIBPUFSADEXAG6082	6x8.25cm			
FIBPUFSADAG6085	FIBPUFSADEXAG6085	6x8.5cm			
FIBPUFSADAG7080	FIBPUFSADEXAG7080	7x8cm			
FIBPUFSADAG6580	FIBPUFSADEXAG6580	6.5x8cm			
FIBPUFSADAG7575	FIBPUFSADEXAG7575	7.5x7.5cm			
FIBPUFSADAG6090	FIBPUFSADEXAG6090	6x9cm			
FIBPUFSADAG60100	FIBPUFSADEXAG60100	6x10cm			
FIBPUFSADAG5082	FIBPUFSADEXAG5082	5x8.25cm			
FIBPUFSADAG75100	FIBPUFSADEXAG75100	7.5x10cm			
FIBPUFSADAG80100	FIBPUFSADEXAG80100	8x10cm			
FIBPUFSADAG82100	FIBPUFSADEXAG82100	8.25x10cm			
FIBPUFSADAG70120	FIBPUFSADEXAG70120	7x12cm			
FIBPUFSADAG80120	FIBPUFSADEXAG80120	8x12cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag adhesive Film Frame Style	Fibrosol® Ag Extra adhesive Film Frame style	cm	inch	Pieces/Box	Boxes/Case
FIBPUFSADAG83120	FIBPUFSADEXAG83120	8.3x12cm			
FIBPUFSADAG90150	FIBPUFSADEXAG90150	9x15cm			
FIBPUFSADAG80150	FIBPUFSADEXAG80150	8x15cm			
FIBPUFSADAG82150	FIBPUFSADEXAG82150	8.25x15cm			
FIBPUFSADAG100140	FIBPUFSADEXAG100140	10x14cm			
FIBPUFSADAG83180	FIBPUFSADEXAG83180	8.3x18cm			
FIBPUFSADAG82200	FIBPUFSADEXAG82200	8.25x20cm			
FIBPUFSADAG90200	FIBPUFSADEXAG90200	9x20cm			
FIBPUFSADAG200200	FIBPUFSADEXAG200200	20x20cm			
FIBPUFSADAG83240	FIBPUFSADEXAG83240	8.3x24cm			
FIBPUFSADAG82250	FIBPUFSADEXAG82250	8.25x25cm			
FIBPUFSADAG90250	FIBPUFSADEXAG90250	9x25cm			
FIBPUFSADAG82300	FIBPUFSADEXAG82300	8.25x30cm			
FIBPUFSADAG90300	FIBPUFSADEXAG90300	9x30cm			
FIBPUFSADAG90350	FIBPUFSADEXAG90350	9x35cm			
FIBPUFSADAG82350	FIBPUFSADEXAG82350	8.25x35cm			
FIBPUFSADAG100350	FIBPUFSADEXAG100350	10x35cm			
FIBPUFSADAG83415	FIBPUFSADEXAG83415	8.3x41.5cm			
FIBPUFSADAG100160	FIBPUFSADEXAG100160	10x16cm			
FIBPUFSADAG100210	FIBPUFSADEXAG100210	10x21cm			
FIBPUFSADAG100260	FIBPUFSADEXAG100260	10x26cm			
FIBPUFSADAG100400	FIBPUFSADEXAG100400	10x40cm			
FIBPUFSADAG120160	FIBPUFSADEXAG120160	12x16cm			
FIBPUFSADAG72250	FIBPUFSADEXAG72250	7.2x25cm			
FIBPUFSADAG5072	FIBPUFSADEXAG5072	5x7.2cm			
FIBPUFSADAG5075	FIBPUFSADEXAG5075	5x7.5cm			
FIBPUFSADAG6075	FIBPUFSADEXAG6075	6x7.5cm			
FIBPUFSADAG70100	FIBPUFSADEXAG70100	7x10cm			
FIBPUFSADAG85155	FIBPUFSADEXAG85155	8.5x15.5cm			
FIBPUFSADAG85150	FIBPUFSADEXAG85150	8.5x15cm			
FIBPUFSADAG8595	FIBPUFSADEXAG8595	8.5x9.5cm			
FIBPUFSADAG90100	FIBPUFSADEXAG90100	9x10cm			
Fibrosol® Ag adhesive Ultra		cm	inch	Pieces/Box	Boxes/Case
FIBULTRAAG5070		5x7cm			
FIBULTRAAG100100		10x10cm			
FIBULTRAAG150150		15x15cm			
FIBULTRAAG100150		10x15cm			
FIBULTRAAG100200		10x20cm			
FIBULTRAAG200450		20x45cm			
FIBULTRAAG100250		10x25cm			
FIBULTRAAG100300		10x30cm			
FIBULTRAAG6070		6x7cm			
FIBULTRAAG100120		10x12cm			
FIBULTRAAG6080		6x8cm			
FIBULTRAAG165180		16.5x18cm			
FIBULTRAAG220220		22x22cm			
FIBULTRAAG5090		5x9cm			
FIBULTRAAG6083		6x8.3cm			
FIBULTRAAG50100		5x10cm			
FIBULTRAAG6082		6x8.25cm			
FIBULTRAAG6085		6x8.5cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag Extra adhesive Ultra		cm	inch	Pieces/Box	Boxes/Case
FIBULTRAEXAG100300		10x30cm			
FIBULTRAEXAG6070		6x7cm			
FIBULTRAEXAG100120		10x12cm			
FIBULTRAEXAG6080		6x8cm			
FIBULTRAEXAG165180		16.5x18cm			
FIBULTRAEXAG220220		22x22cm			
FIBULTRAEXAG60100		6x10cm			
FIBULTRAEXAG100350		10x35cm			
FIBULTRAEXAG200200		20x20cm			
FIBULTRAEXAG100160		10x16cm			
FIBULTRAEXAG100210		10x21cm			
FIBULTRAEXAG100260		10x26cm			
FIBULTRAEXAG100400		10x40cm			
FIBULTRAEXAG120160		12x16cm			
FIBULTRAEXAG72250		7.2x25cm			
FIBULTRAEXAG5072		5x7.2cm			
FIBULTRAEXAG5075		5x7.5cm			
FIBULTRAEXAG6075		6x7.5cm			
FIBULTRAEXAG6085		6x8.5cm			
FIBULTRAEXAG70100		7x10cm			
FIBULTRAEXAG80100		8x10cm			
FIBULTRAEXAG85155		8.5x15.5cm			
FIBULTRAEXAG85150		8.5x15cm			
FIBULTRAEXAG8595		8.5x9.5cm			
FIBULTRAEXAG90100		9x10cm			
Fibrosol® Ag IV	Fibrosol® Ag Extra IV	cm	inch	Pieces/Box	Boxes/Case
FIBIVAG6070	FIBIVEXAG6070	6x7cm			
FIBIVAG100100	FIBIVEXAG100100	10x10cm			
FIBIVAG150150	FIBIVEXAG150150	15x15cm			
FIBIVAG100150	FIBIVEXAG100150	10x15cm			
FIBIVAG100200	FIBIVEXAG100200	10x20cm			
FIBIVAG200450	FIBIVEXAG200450	20x45cm			
FIBIVAG5070	FIBIVEXAG5070	5x7cm			
FIBIVAG100250	FIBIVEXAG100250	10x25cm			
FIBIVAG100300	FIBIVEXAG100300	10x30cm			
FIBIVAG100120	FIBIVEXAG100120	10x12cm			
FIBIVAG6080	FIBIVEXAG6080	6x8cm			
FIBIVAG165180	FIBIVEXAG165180	16.5x180cm			
FIBIVAG220220	FIBIVEXAG220220	22x22cm			
FIBIVAG7080	FIBIVEXAG7080	7x8cm			
FIBIVAG82100	FIBIVEXAG82100	8.25x10cm			
FIBIVAG7085	FIBIVEXAG7085	7x8.5cm			
FIBIVAG7090	FIBIVEXAG7090	7x9cm			
FIBIVAG5057	FIBIVEXAG5057	5x5.7cm			
FIBIVAG7070	FIBIVEXAG7070	7x7cm			
FIBIVAG80100	FIBIVEXAG80100	8x10cm			
FIBIVAG6085	FIBIVEXAG6085	6x8.5cm			
FIBIVAG85100	FIBIVEXAG85100	8.5x10cm			
FIBIVAG85115	FIBIVEXAG85115	8.5x11.5cm			
FIBIVAG90100	FIBIVEXAG90100	9x10cm			
FIBIVAG100140	FIBIVEXAG100140	10x14cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag IV	Fibrosol® Ag Extra IV	cm	inch	Pieces/Box	Boxes/Case
FIBIVAG100155	FIBIVEXAG100155	10x15.5cm			
FIBIVAG5055	FIBIVEXAG5055	5x5.5cm			
FIBIVAG6570	FIBIVEXAG6570	6.5x7cm			
Fibrosol® Ag IV Film	Fibrosol® Ag Extra IV Film	cm	inch	Pieces/Box	Boxes/Case
FIBIVPUAG6070	FIBIVPUEXAG6070	6x7cm			
FIBIVPUAG100100	FIBIVPUEXAG100100	10x10cm			
FIBIVPUAG150150	FIBIVPUEXAG150150	15x15cm			
FIBIVPUAG100150	FIBIVPUEXAG100150	10x15cm			
FIBIVPUAG100200	FIBIVPUEXAG100200	10x20cm			
FIBIVPUAG200450	FIBIVPUEXAG200450	20x45cm			
FIBIVPUAG5070	FIBIVPUEXAG5070	5x7cm			
FIBIVPUAG100250	FIBIVPUEXAG100250	10x25cm			
FIBIVPUAG7070	FIBIVPUEXAG7070	7x7cm			
FIBIVPUAG100120	FIBIVPUEXAG100120	10x12cm			
FIBIVPUAG6080	FIBIVPUEXAG6080	6x8cm			
FIBIVPUAG165180	FIBIVPUEXAG165180	16.5x180cm			
FIBIVPUAG220220	FIBIVPUEXAG220220	22x22cm			
FIBIVPUAG7080	FIBIVPUEXAG7080	7x8cm			
FIBIVPUAG82100	FIBIVPUEXAG82100	8.25x10cm			
FIBIVPUAG7085	FIBIVPUEXAG7085	7x8.5cm			
FIBIVPUAG7090	FIBIVPUEXAG7090	7x9cm			
FIBIVPUAG5057	FIBIVPUEXAG5057	5x5.7cm			
FIBIVPUAG80100	FIBIVPUEXAG80100	8x10cm			
FIBIVPUAG6085	FIBIVPUEXAG6085	6x8.5cm			
FIBIVPUAG85100	FIBIVPUEXAG85100	8.5x10cm			
FIBIVPUAG90100	FIBIVPUEXAG90100	9x10cm			
FIBIVPUAG100140	FIBIVPUEXAG100140	10x14cm			
FIBIVPUAG85115	FIBIVPUEXAG85115	8.5x11.5cm			
FIBIVPUAG100300	FIBIVPUEXAG100300	10x30cm			
Fibrosol® Ag Foam Gentle	Fibrosol® Ag Extra Foam Gentle	cm	inch	Pieces/Box	Boxes/Case
FIBFOAMGAG5050	FIBFOAMGEXAG5050	5x5cm			
FIBFOAMGAG100100	FIBFOAMGEXAG100100	10x10cm			
FIBFOAMGAG150150	FIBFOAMGEXAG150150	15x15cm			
FIBFOAMGAG100150	FIBFOAMGEXAG100150	10x15cm			
FIBFOAMGAG100200	FIBFOAMGEXAG100200	10x20cm			
FIBFOAMGAG200450	FIBFOAMGEXAG200450	20x45cm			
FIBFOAMGAG5070	FIBFOAMGEXAG5070	5x7cm			
FIBFOAMGAG100250	FIBFOAMGEXAG100250	10x25cm			
FIBFOAMGAG100300	FIBFOAMGEXAG100300	10x30cm			
FIBFOAMGAG6070	FIBFOAMGEXAG6070	6x7cm			
FIBFOAMGAG100120	FIBFOAMGEXAG100120	10x12cm			
FIBFOAMGAG6080	FIBFOAMGEXAG6080	6x8cm			
FIBFOAMGAG165180	FIBFOAMGEXAG165180	16.5x18cm			
FIBFOAMGAG220220	FIBFOAMGEXAG220220	22x22cm			
FIBFOAMGAG75100	FIBFOAMGEXAG75100	7.5x10cm			
FIBFOAMGAG100350	FIBFOAMGEXAG100350	10x35cm			
FIBFOAMGAG200200	FIBFOAMGEXAG200200	20x20cm			
FIBFOAMGAG125125	FIBFOAMGEXAG125125	12.5x12.5cm			
FIBFOAMGAG170170	FIBFOAMGEXAG170170	17x17cm			
FIBFOAMGAG6060	FIBFOAMGEXAG6060	6x6cm			
FIBFOAMGAG100125	FIBFOAMGEXAG100125	10x12.5cm			

Fibrosol® Ag Family Dressings


Fibrosol® Ag Foam Gentle	Fibrosol® Ag Extra Foam Gentle	cm	inch	Pieces/Box	Boxes/Case
FIBFOAMGAG175175	FIBFOAMGEXAG175175	17.5x17.5cm			
FIBFOAMGAG7575	FIBFOAMGEXAG7575	7.5x7.5cm			
FIBFOAMGAG150200	FIBFOAMGEXAG150200	15x20cm			
Fibrosol® Ag Foam Basic	Fibrosol® Ag Extra Foam Basic	cm	inch	Pieces/Box	Boxes/Case
FIBFOAMBAG100100	FIBFOAMBEXAG100100	10x10cm			
FIBFOAMBAG100200	FIBFOAMBEXAG100200	10x20cm			
FIBFOAMBAG150150	FIBFOAMBEXAG150150	15x15cm			
FIBFOAMBAG200200	FIBFOAMBEXAG200200	20x20cm			
FIBFOAMBAG165180	FIBFOAMBEXAG165180	16.5x18cm			
FIBFOAMBAG220220	FIBFOAMBEXAG220220	22x22cm			
FIBFOAMBAG5050	FIBFOAMBEXAG5050	5x5cm			
FIBFOAMBAG7575	FIBFOAMBEXAG7575	7.5x7.5cm			
FIBFOAMBAG75100	FIBFOAMBEXAG75100	7.5x10cm			
FIBFOAMBAG100150	FIBFOAMBEXAG100150	10x15cm			
FIBFOAMBAG100250	FIBFOAMBEXAG100250	10x25cm			
FIBFOAMBAG100300	FIBFOAMBEXAG100300	10x30cm			
FIBFOAMBAG150200	FIBFOAMBEXAG150200	15x20cm			
FIBFOAMBAG100350	FIBFOAMBEXAG100350	10x35cm			
FIBFOAMBAG6060	FIBFOAMBEXAG6060	6x6cm			
FIBFOAMBAG100125	FIBFOAMBEXAG100125	10x12.5cm			
FIBFOAMBAG125125	FIBFOAMBEXAG125125	12.5x12.5cm			
FIBFOAMBAG170170	FIBFOAMBEXAG170170	17x17cm			
FIBFOAMBAG175175	FIBFOAMBEXAG175175	17.5x17.5cm			
Fibrosol® Ag OTC Bandage		cm	inch	Pieces/Box	Boxes/Case
FIBOTCAG938		0.9x3.8cm			
FIBOTCAG1938		1.9x3.8cm			
FIBOTCAG1972		1.9x7.2cm			
FIBOTCAG2572		2.5x7.2cm			
FIBOTCAG25		Ø2.5cm			
FIBOTCAG2525		2.5x2.5cm			
FIBOTCAG5072		5x7.2cm			
FIBOTCAG5060		5x6cm			
FIBOTCAG4563		4.5x6.3cm			
FIBOTCAG50100		5x10cm			
FIBOTCAG1657		1.6x5.7cm			
FIBOTCAG1964		1.9x6.4cm			
FIBOTCAG1976		1.9x7.6cm			
FIBOTCAG2576		2.5x7.6cm			
FIBOTCAG3055		3x5.5cm			
FIBOTCAG3064		3x6.4cm			
FIBOTCAG3072		3x7.2cm			
FIBOTCAGANCHOR		3x7.2cm			
FIBOTCAG3838		3.8x3.8cm			
FIBOTCAG3872		3.8x7.2cm			
FIBOTCAG22		Ø2.2cm			
FIBOTCAGFEET		FEET			
FIBOTCAG2235		2.2x3.5cm			
FIBOTCAG2665		2.6x6.5cm			
FIBOTCAG3070		3x7cm			
FIBOTCAG64110		6.4x11cm			
FIBOTCAG5284		5.2x8.4cm			

Fibrosol® Ag Family Dressings

Fibrosol® Ag OTC Bandage	cm	inch	Pieces/Box	Boxes/Case
FIBOTCAG20120	2x12cm			
FIBOTCAG30120	3x12cm			
FIBOTCAG20180	2x18cm			
FIBOTCAG30180	3x18cm			
FIBOTCAG2582	2.5x8.2cm			
FIBOTCAG2956	2.9x5.6cm			
FIBOTCAGCHARACTERS	1.9x6.3cm			
FIBOTCAG1955	1.9x5.5cm			
FIBOTCAG4550	4.5x5cm			
Fibrosol® OTC Dressing strip Ag	cm	inch	Pieces/Box	Boxes/Case
FIBDERMIAG60100	6x10cm			
FIBDERMIAG80100	8x10cm			
FIBDERMIAG601000	6x100cm			
FIBDERMIAG801000	8x100cm			

- Amria - Alexandria - Egypt
 Tel. : 002 03 4500264 Fax : 002 03 4500263
 - Kafr-El-Zayat P.O.Box 38 Egypt
 Tel. : 002 040 2545643 Fax : 002 040 2545645
 E-mail : sales@pharmaplast-online.com

 **Pharmaplast**
 www.pharmaplast-online.com

Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast		Product name : Fibrosol Ag family						
Issue date : 22-02-2019		Prepared by : Nader Nabil zaki						
Colors : 		Dimension : A4						
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	13-04-2021	new codes	file maintenance					
3	17-01-2022	new codes	file maintenance					
4	03-01-2023	adding new product	file maintenance					